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Putting a mask on your face.

Hold the hook with your right hand, hang your finger on the opposite rubber band with your left hand, put the mask on your face.

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Hooking the rubber band.

Bring the rubber band through the upper and lower of ears and hook the left and right rubber bands at the back of the head. Make sure there are no twists in the rubber band.

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Adjusting the length of rubber band.

While holding the mask with one hand, adjust the length by pulling with the end of the lower rubber band. Then adjust the length of the upper rubber band as well.

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Making the nosepiece into a nose shape.

Make the nosepiece into a nose shape with your fingers and bring it close.

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Checking the seal of the mask and your face.

Put both hands on the mask, exhale and check for air leaks through the gap between face and mask.

If there is air leakage, adjust the position of the mask and the length of the rubber band and bring it close to the face.

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Positioning of rubber band on head.

Put a mask on your face. Pull the lower band over your head and position it around the neck first, and then pull the upper band over your head and position it around the back of your head.

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